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Ordering information

345103/10 Pack size: 10 items
345103/50 Pack size: 50 items

Medical Product Class I
For reduction of the transmission of pathogens through droplet infection

Properties and intended use

Size 17.5 x 9.5 cm, 3-ply soft fleece, hypoallergenic, non-sterile, white inside, blue outside, a micro germ filter in between, nose piece, elastic ear straps for securing. Latex-free. Nose piece nickel-free. BFE > 98 %. DIN EN 14683:2019 Type IIR.

The non-sterile mask is a disposable product and serves primarily to protect the environment from aerosols and droplets that may contain pathogens when exhaling, but also reduces the pathogen load of the inhaled air.

Storage and shelf life

Masks can be stored in a sealed, undamaged and clean box at room temperature until the expiry date. Protect from moisture, chemicals and direct sunlight.

Using the product

1. Before using the mask, clean hands with soap and water or disinfectant.
2. Hold the mask by the ear straps so that the blue side is facing outwards and the nose piece is facing upwards, and put it over the nose with the nose piece.
3. Place the ear straps behind the ears.
4. Press the nose piece onto the nose and pull the mask over the chin.
5. The chin, mouth and nose must be covered by the mask. Make sure there are minimal gaps between the face and the mask.
6. Do not touch the mask during use.
7. Replace soaked masks with a new one.
8. Maximum wearing time 8 hours.
9. To remove the mask, grasp the ear straps.
10. Dispose of the mask after removing and clean hands (see 1.)

Cautions

The medical face mask is a single-use medical device.

Check the pack contents for completeness before use; do not use damaged packs or packs after the expiry date printed on them.

Information on safe disposal

Dispose of soaked or used face masks in the residual waste.

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Albert Kerbl GmbH
Kerbl UK Ltd Lands End Way