-  *Bedienungsanleitung*
-  *Instruction Manual*
-  *Manual de instrucciones*
-  *Mode d'emploi*

ECO-MATIC® TWIN

*mit Flaschenaufsatz & Schlauchansatz
with bottle attachment & tube connector
con adaptador para frascos & tubo
avec porte-bouteille & embase de tuyau*

by HSW for veterinary use only

Barrel sizes 0,5ml, 1 ml or 2 ml – fixed doses

*Henke-Sass, Wolf GmbH
Keltenstrasse 1
D-78532 Tuttlingen
Germany
Tel. +49 74 62 94 66 207
Fax +49 74 62 94 66 52 08
www.henkesasswolf.de
E-mail: info@henkesasswolf.de*

ECO-MATIC® TWIN

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
ECO-MATIC® TWIN

- *Automatic self-filling syringe*
- *with bottle mount and tubing attachments*

- A) Ergonomic handle
- B) Plastic barrel with bottle mount attachment
- C) Adjustable needle nut for needle distance adjustment
- D) Adjustable barrel collar
- E) Metal needle fitting, luer lock
- F) 2 x universal bottle collar Ø 20mm
- G) 2 x universal bottle collar Ø 30mm
- H) 2 x tubing collar w. spigot Ø 4-6mm
- I) 2 x PVC tubing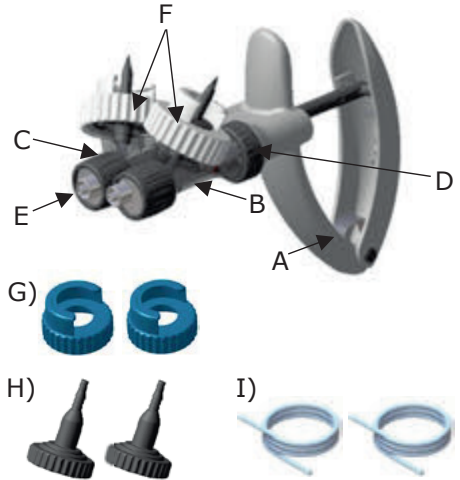

The HSW ECO-MATIC® TWIN self-filling dosing syringe is designed for continuous use, the O-ring, barrel and tubing can be replaced if necessary.


The HSW ECO-MATIC® TWIN can be used in multiple set ups to accommodate all bottle types and sizes. Any combination of bottle mount basket, bottle mount collar and tubing are possible.

It is essential that the syringe is cleaned before each use. Please follow the manufacturers' recommendations with any cleaning or disinfection solutions used.

HSW ECO-MATIC® TWIN Set Up


1) Positioning the Barrel

- Turn the adjustable barrel collar (D) while holding the barrel to loosen.
- Once loose, you can adjust the barrel position as required.
- To tighten, hold the barrel in position and tighten the barrel collars.
- Important! When using the syringe as a bottle mount the spike and bottle must always face upwards. With a draw off set up, the tubing collar can be upward or downward facing.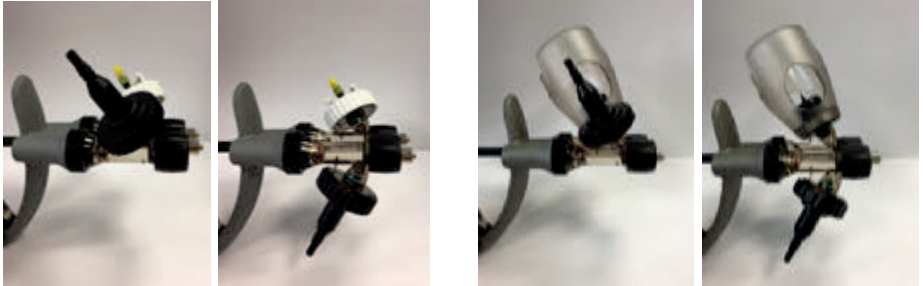

2) Setting the Needle Distance

- The needle distance can be changed based on the age, size and type of animal. Please consult with your veterinarian for recommendations.
- There are seven needle distance settings between 20mm and 36mm possible.
- Loosen the needle nut (C).
- Take the needle nut off and position the metal needle fitting in one of the seven positions and replace.
- Tighten the needle nut.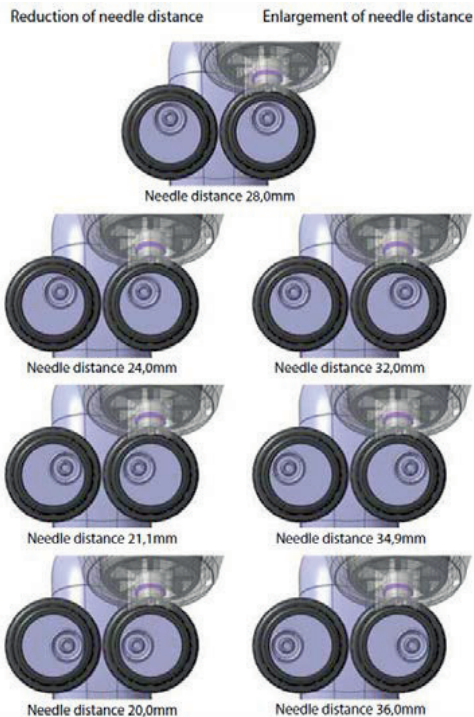


3) **TO FIT THE NEEDLE**

Attention:

- For your safety: Leave the protective cover (disposable needles) on the needle until ready for use.
- Insert the needle onto the luer lock fitting and secure the needle clockwise.
- Disconnect the needle fitting the opposite way.

4) **DRAW-OFF SET UP**

- Attach the tubing to the spigot collar. Push the tube on to the spigot as much as possible to insure a tight connection.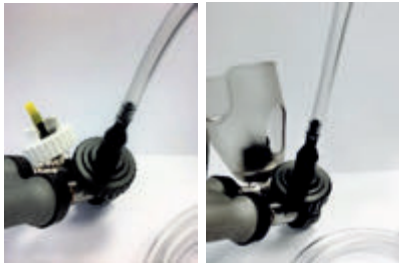


With a bottle mount tubing combination it is recommended to partially prime the tubing side first, than afterwards the bottle mount side. This will avoid product waste on the bottle mount side during the priming of the tubing.

5) **BOTTLE MOUNT SET UP**

It is recommended to make a pilot hole in the rubber cap of the bottle prior to connecting to the syringe spike. Use a sterile needle to puncture the rubber cap than guide the spike thru this hole.


a. Universal Bottle Collar set up

- There are two sizes of collars, white Ø 20mm and blue Ø 30mm.
- Select the corresponding collar size to fit your bottle.
- Guide the spike in to the pilot hole in the center of the bottle.
- Turn the collar until the bottle fits securely on the bottle mount. (Please note: Don't turn the bottle itself, only the collar!)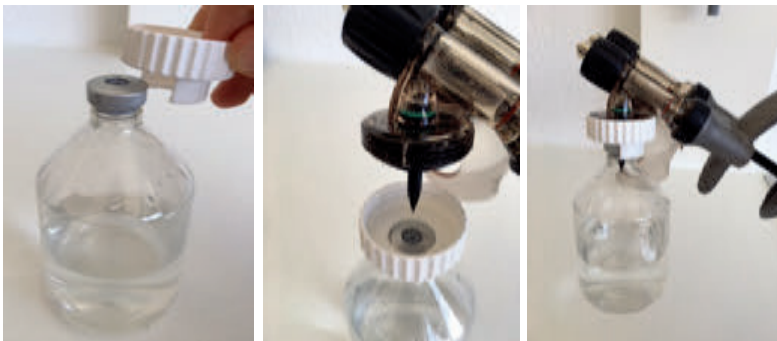

b. Basket (sold separately)

- Insert the bottle into the basket.
- Line up the bottle, by using the viewing window of the basket, in the centre of the rubber cap and pierce the spike into the bottle while rotating the bottle slightly.

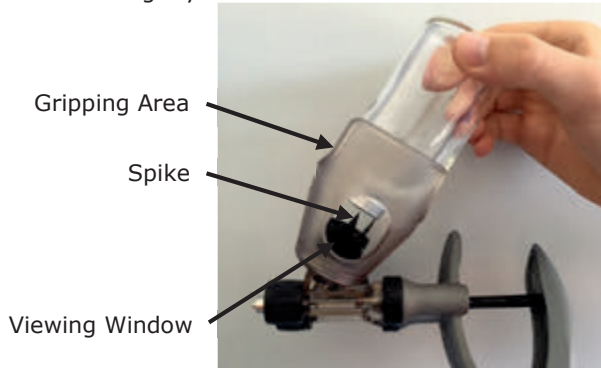- To Pull out the bottle, use the gripping area on both sides of the bottle.

After choosing one of the above adapters begin with priming:

- Remove protection cover of the needle.
- Hold the ECO-MATIC® at a slight incline and take care that the spike is below the liquid to avoid sucking air in to the barrel.
- Squeeze the adjustable handle and then release quickly to draw fluid into the barrel.
- To remove any air bubbles inside the barrel, repeat step (3).

6) INSTRUCTIONS FOR AFTER USE

Remove all traces of material from syringe by flushing thoroughly with a hot water detergent mix. Rinse thoroughly with clean, warm water.

Attention!

If you detect that the syringe draws air after the described cleaning, it is possible that the valve is fixed in the front metal approach. The valve can be released by knocking slightly the front of the syringe against an object, e.g., the edge of a table, and afterwards rinsing the syringe with warm water.

7) DISINFECTION

It is recommended that the HSW ECO-MATIC® and a supply of needles or nozzles be disinfected before and after each use.

1. Suck „hot“ water into the barrel by using an attached bottle with warm water.
2. Suspend the complete HSW ECO-MATIC® including needles or nozzles in a container of water and boil it for at least 20 minutes. Ideally, the cylinder will be disassembled from the handle and the syringe will be boiled in a disassembled condition.
3. Remove the HSW ECO-MATIC® from container, wrap cloth around the handle and pump it dry.

8) TO LUBRICATE:

Disconnect plastic barrels from the handle by removing adjustable barrel collars (D) and sliding barrels off syringe body. Lubricate piston and glass barrel with 100% pure oil, cold squeezed or equivalent vegetable oil.

Please follow the manufacturer's durability and storage life instruction. Do not use mineral oil, silicone, WD-40, glycerine or alcohol. Petroleum based products harden the piston, reducing the life of the component.

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© copyright · Henke-Sass, Wolf GmbH · 8300035540- 10/18 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 8300035540- 10/18 · Subject to amendments in technology and design · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 8300035540- 10/18 · Reservado a los derechos de modificaciones técnicas y diseño · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 8300035540- 10/18 · Sous réserve de modifications dans le design et la technique · Printed in Germany