-  *Bedienungsanleitung*
-  *Instruction Manual*
-  *Manual de instrucciones*
-  *Mode d'emploi*

## *HENKE-JECT® TBC*

by **HSW**

*for veterinary use only*

**2,0 ml**

Henke-Sass, Wolf GmbH  
Keltenstrasse 1  
D-78532 Tuttlingen  
Germany  
Tel. +49 74 62 94 66 207  
Fax +49 74 62 94 66 52 08  
[www.henkesasswolf.de](http://www.henkesasswolf.de)  
E-mail: [info@henkesasswolf.de](mailto:info@henkesasswolf.de)

**HENKE-JECT® TBC**

von **HSW**

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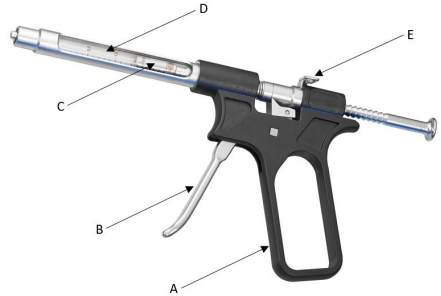
- A) ergonomic designed handle**
- B) trigger**
- C) high-precision metal piston**
- D) glass barrel**
- E) ratchet lock**

The HENKE-JECT® dosage syringe has been developed and designed for the specific requirement of tuberculosis vaccination and for continuous use.

The syringe can be disassembled very easy for cleaning.

**It is essential that the syringe is cleaned and disinfected before each use. Please follow the manufacturers' recommendations with any cleaning or disinfectant solutions used.**

### **TO FIT THE NEEDLE**

#### **Attention:**

**For your safety: Leave the protective cover (only on disposable needles) on the needle until the syringe is ready for use.**

- Insert needle onto Luer Lock fitting and secure the needle clockwise.
- disconnect the needle fitting the opposite way.

### **DRAW Medicament**

- 1) Press ratchet lock which will release the piston to be moved and insert the needle into the bottle.
- 2) Pull piston backwards until stop of the piston is reached.  
The syringe is now ready for use.
- 3) One complete trigger pull releases 0,1 ml of the medicament.

1

2

3

### **INSTRUCTION AFTER USE OF THE HENKE-JECT® TBC**

Remove all traces of material from syringe by flushing thoroughly with hot water detergent mix. Rinse thoroughly with clean, warm water.

### **Disinfection**

- 1) Clean the syringe under hot water.
- 2) Suspend the complete HENKE-JECT® TBC including needles in a container of water and boil the same for at least 20 minutes.
- 3) Remove then syringe from the container, wrap cloth around the handle and pump it dry.

**Attention:** Calcium-containing water can lead to deposits on the piston and thus damage the syringe.

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© **copyright** · **Henke-Sass, Wolf GmbH** · 83000006362-10/16 · Änderungen in Technik und Design vorbehalten · Gedruckt in Deutschland
© **copyright** · **Henke-Sass, Wolf GmbH** · 83000006362-10/16 · Subject to amendments in technology and design · Printed in Germany
© **copyright** · **Henke-Sass, Wolf GmbH** · 83000006362-10/16 · Reservado los derechos de modifi caciones técnicas y de diseño · Impreso en Alemania
© **copyright** · **Henke-Sass, Wolf GmbH** · 83000006362-10/16 · Sous réserve de modifi cations dans le design et la technique · Imprimé en Allemagne